

DENTAL, INC.

5860 FLYNN CREEK ROAD P.O. BOX 106 COMPTCHE, CA USA 95427 PHONE (707) 937-0521 1 800 233-0521 FAX (707) 937-2809






INSTALLATION INSTRUCTIONS **COLLET SHIELD RH** Product No. Q104

OVERVIEW 1.0


The non-rotating collet shield covers the collet to provide a steady rest for more comfortable operation of your WELLS Super Quick Chuck. It also helps protect rubber gloves. Removal is easy for cleaning around the collet. Q104 COLLET SHIELD RH (right hand) fits model Q015 WELLS SUPER QUICK CHUCK (right hand with knurled dust cover). See figure 1. If there is no knurled dust cover, the collet shield cannot be installed.

2.0 **INSTALLATION**


- 2.1 Remove the knurled dust cover by rotating CLOCKWISE. See Figure 2. It has left-hand threads.
- 2.2 Clean around the collet, collet body and threads with a small brush or rag. See Figure 3.
- 2.3 Screw the collet shield onto the threads rotating COUNTERCLOCKWISE. See Figure 4. Tighten firmly by hand.

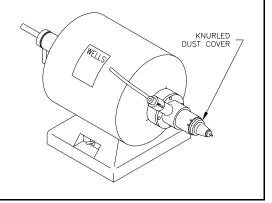




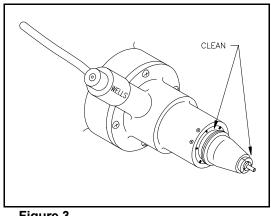



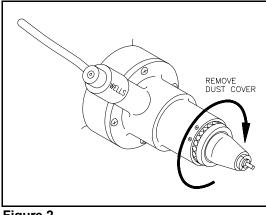



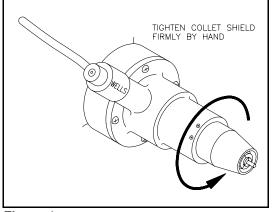

3.0 MAINTENANCE

The collet shield should be removed periodically so accumulated material can be removed from around the collet, collet body and threads.

- Remove the collet shield by rotating CLOCKWISE. Be sure to 3.1 rotate in the correct direction. It has left-hand threads.
- Clean around the collet, collet body and threads with a small brush 3.2 or raq.
- 3.3 Screw the collet shield onto the threads by rotating COUNTERCLOCKWISE. See Figure 4. Tighten firmly by hand.

Figure 4