ANNUAL CATALOG 2022

Dental Laboratory Equipment And Supplies



WELLS DENTAL, INC.

ORDERING & CATALOG INFORMATION

TECHNICAL ASSISTANCE

If you want information about our products or need help to install or service your equipment, please call us. One of our experienced technicians will give you courteous, professional advice.

FACTORY REPAIR SERVICE

Since 1936, Wells Dental's multifaceted factory service program has substantially saved our customers money while restoring the quality of their Wells products. For more information on Wells repair services, see the last section of this catalog.

EQUIPMENT EXCHANGE

If you cannot be without your equipment for the one or two days it takes us to repair your item our exchange service is available. We can send you a rebuilt item with the understanding that you will return your old item to us. Enclosed with the packing list on top of the box you will find a UPS prepaid RS (return service) label. Simply enclose your old equipment in the same box and affix the prepaid return label. You can give the package to any UPS driver and the package will be returned to us. The equipment exchanged will be invoiced at full retail price which serves as a deposit. Upon receipt of your old item, appropriate credit will be issued for the difference between the retail price and the exchange price. No credit will be issued for exchanges not returned within 30 days of invoice date.

PRICE & PRODUCT CHANGES

We reserve the right to change prices, impose purchase restrictions and to make product or engineering changes at any time without prior notification.

WARRANTIES

Beginning at the date of purchase, Wells equipment is warranted for the indicated periods and subject to the following conditions:

Limited, 1-year prorated*

New and rebuilt Wells Handpieces

Foot Controls

Super Quick Chucks

High Speed Spindles

1 year — New Engine Unit motors

5 years — New Whisper Arms

All warranties and guarantees are void if the equipment has been oiled or subjected to tampering or abuse. Charges for services by others will not be reimbursed unless previously authorized in writing.

*Prorated, for terms of all Wells warranties, means:

After 30 days from the date of purchase, one twelfth of the current rebuild price will be charged per month of use up to a maximum of twelve months plus shipping.

RETURN OF GOODS

No return authorization needed. Please ship unused goods in the original packaging and enclose a copy of the invoice. Refund is subject to the condition of the goods, shipping and handling charges. Special orders are not returnable.

ORDERING

Call or	fax toll-free	Hours:
Phone:	1-800-233-052	
Fax:	1-800-809-755	Pacific Time
URL:	www.wellsden	tal.com
Mail:	Wells Dental, Ir PO Box 106 5860 Flynn Cree Comptche, CA 95427- 0106	ek Rd.

PAYMENT

Terms are NET 30 days from date of invoice, on approved credit. A late fee of 1.5% per month (18% annual rate) will be charged on overdue invoices.

We accept MasterCard, Visa, American Express and Discover for advance and C.O.D. orders.

SHIPPING

Items are normally shipped UPS Ground within 24 hours. Shipments are F.O.B. Comptche, CA. They become the responsibility of the customer when the carrier accepts the goods. Inspect the shipment upon arrival and report any damages or shortages directly to the carrier. Although we do our best to meet the delivery dates supplied by our

representatives, they do not constitute a guarantee. For estimated delivery time to your location, please inquire when ordering. At your request, we can ship next day (not guaranteed from our location), second day or 3 day UPS delivery.



Wells Dental

today, located in the heart of the California Redwoods.

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

ENGINE UNITS & COMPONENTS

Quiet soft action, powerful and easy to maintain, Wells engine units set the standard for quality.

Put precise power at your finger tips with a Wells complete laboratory engine unit. Tough stainless steel whisper arms move effortlessly on permanently lubricated ball bearings to ensure the smoothest motion. Choose tough Pyrathane or economical cotton drive belts to deliver the power of 115V or 230V motor to the precision handpiece. The foot control frees your hands for more important tasks.

ENGINE UNIT SPECIFICATIONS

Collet size:
Maximum forward speed:
Maximum reverse speed:
Electrical rating:
Shipping weight:
*Also available:

U900

3/32" 24,000 RPM 20,000 RPM 115 V, 60 Hz, single-phase, 1.9 Amp* 21 lbs 230V, 60 Hz, single-phase, 1.0 Amp

\$1434.00



Engine unit shown with chrome hood.

See next page for foot control information.


1

WELLS DENTAL, INC.

	 Hammered gray painted hood 	
U903	Engine Unit Chrome Long Arm • Chrome plated hood	N/A

Engine Unit GrayLong Arm

U901	Engine Unit Gray Short Arm • Hammered gray painted hood	1434.00
U904	Engine unit Chrome Short Arm • Chrome plated hood	N/A

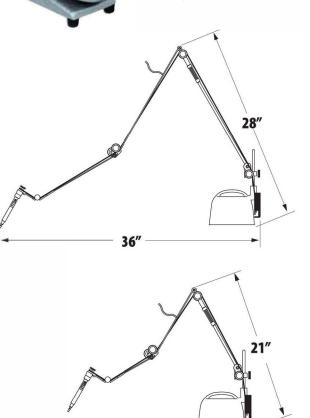

Toll Free FAX within U.S. 1-800-809-7553

VOICE outside U.S. FAX


707-937-0521

24″

707-937-2809

H010

LABORATORY HANDPIECE & WRIST JOINT

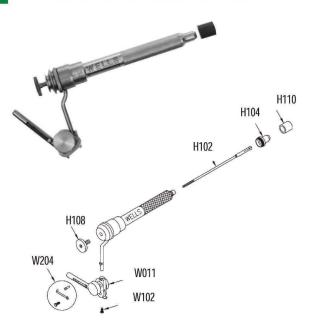

	Permanently lubricated ball bearings Stainless steel body Precision collet	
REPLACE	MENT PARTS	
H108	Thumbscrew	\$19.00
H102	Collet	\$28.00
H104	Nose Cone	\$11.00
H106	Nose Cone Contra Angle	\$15.00
H110	Rubber Wrench	\$3.00
H206	Collet & Thumbscrew	\$43.00
H208	Nose Cone with Rubber Wrench	\$12.00
W011	Wrist Joint	\$16600
W204	Spring Clip Kit	\$8.00
W102	Retaining Screw Pkg-2	\$5.00

Handpiece with Wrist Joint

FOOT CONTROL & REPLACEMENT PARTS

For replacement parts not listed in this catalog, please refer to your User's Manual or give us a call.



Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

\$450.00

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

ENGINE UNITS & COMPONENTS

ENGINE ARMS


AS052 Whisper arm long

- AS051 Whisper arm short
 - 302 stainless spring steel construction of small diameter arms and hook
 - 303 stainless steel construction of large diameter arm, post, block and pulleys
 - · Permanently lubricated ball bearings



ARM PULLEYS

AS211	Whisper arm pulley w/wrench Stainless steel construction 	\$54.00
	 For AS051 (short) and AS052 (long) Whisper A 	Arms only
A106	Aluminum pulley w/wrench	\$42.00
	 For A011 (Piccolo) and A012 (Standard) 	

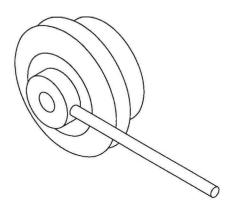

BELTS

AS404	Pyrathane belt 6' 6"For AS051 Whisper Arm Short and A011Engine Arm Piccolo	\$17.00	A400	Cotton belt 6' 6" — Pkg. of 2 • For AS051 Whisper Arm Short and A011 Engine Arm Piccolo	\$22.00
AS406	Pyrathane belt 9'8"For AS052 Whisper Arm Long and A012 Engine Arm Standard	\$17.00	A402	Cotton belt 9' 8" — Pkg. of 2 • For AS052 Whisper Arm Long and A012 Engine Arm Standard	\$22.00

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

ENGINE UNITS & COMPONENTS

ŀ				E026	Hood & Motor Chrome 115V • 115 V, 60 Hz, Single-phase, 1.9 A motor	NA
Ca	ancer and Reproductive Harm www.P65Warnings.ca.gov			E027	 Chrome plated hood Hood & Motor Gray 115V 115 V, 60 Hz, Single-phase, 1.9 A motor Hammered gray painted hood 	\$549.00
				E028	Hood & Motor Chrome 230V • 230 V, 60 Hz, Single-phase, 1.0 A motor • Chrome plated hood	NA
	Shown with hammered	gray painted hood.		E029	Hood & Motor Gray 230V • 230 V, 60 Hz, Single-phase, 1.0 A motor • Hammered gray painted hood	\$510.00
Ca	NOTOR, BRUSHES & REPAIR	PARTS				
E 13 E 13		\$369.00 \$369.00	E101	• 1/8″ x	Baldor Motor — Pkg. of 2 3/16" x 5/8" (3.17mm x 4.76mm x 15.87mm) 32 Baldor Motor	\$22.00
	O FUILLE					
E10	 Brushes Bodine Large — Pkg. of 2 1/4" x 3/8" x 1" (6.35mm x 9.53mm) For Large Brush E131 Motor 	\$22.00 25.4mm)	E107	• 3/16":	Bodine Small — Pkg. of 2 x 1/4″ x 11/16″ (4.76mm x 6.35mm x 17.46mm) all Brush E129 and E130 Motor	\$22.00
	CHAMBER]				
E	 Brushes WSM Motor — Pkg. of 2 1/4" x 1/4" x 3/4" (6.35mm x 6.35 For WSM motors (manufactured beta) 		E150	Motor Pu	lley with Set Screw	\$19.00
		Z		\sim		-
E1	12 Power Cord Kit \$23.00 For replacement pa	E110 5-wire Cord Ki rts not listed in this catalog,			E504 Connector 6-prong Female er's Manual or give us a call.	\$22.00

Toll Free VOICE 1-800-233-0521 within U.S. Toll Free FAX 1-800-809-7553 within U.S. www.wellsdental.com Business Hours: Monday - Friday, 8am - 4:30pm VOICE 707-937-0521 outside U.S.

FAX 707-937-2809 outside U.S.

4

Quiet operation. Fully adjustable for comfort. Jewelers taper on the left hand side.


GOLD CHROME FINISHING MACHINE

U712 Gold Chrome Machine 115V Hz 60Hz \$1839.00

- Wells High Speed Spindle Quick tool changes
 while lathe is running
- Wells Light Shield with tempered glass
- Wells 115 Volt, 1/3 Hp lathe
- Collet size: standard 3/32" (1/16", 1/8" or 3mm collets are available)
- Spindle RPM: 12000 (low) and 24000 (high)
- Lathe RPM: 1725 (low) and 3450 (high)
- Electrical rating: 115 Volt, 60 Hz, single phase, 3.3 Amp
- Shipping weight: 55 lbs.

Also available with 100 volt or 230 volt lathe. 50 Hz or 60 Hz



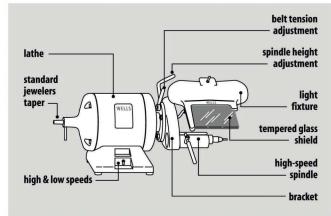

GOLD CHROME BRACKET ASSEMBLY /RW

U011

Gold Chrome Bracket Assembly/RW

• Without lathe for field installation on a Red Wing Lathe

\$959.00

- Collet size: standard 3/32" (1/16", 1/8" or 3mm collets are available.)
- Shipping weight: 9lbs.



Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

5

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

WELLS HIGH SPEED SPINDLES FOR DEMCO & TICONIUM HIGH SPEED ALLOY GRINDERS, BELT DRIVEN

	5020	For Demco E96 High Speed Alloy Grinder • Standard shaft length • S116 with two pulley diameters for double vee belt • S141 Pulley Shield • Two 10-32 threaded mounting holes	\$396.00
	S021	For Demco and Ticonium High Speed Alloy Grinders • Short Shaft • S118 with one pulley diameter for double vee belt • S141 Pulley Shield • Two 10-32 threaded mounting holes	\$396.00
	S018	For Demco B1 High Speed Alloy Grinder • Short shaft • S118 with one pulley diameter for double vee belt • S141 Pulley Shield • S230 Smooth Mounting Tubes	\$396.00
	S019	For Demco High Speed Alloy Grinder • S120 Pulley for flat belt • S141 Pulley Shield • Two 10-32 threaded mounting holes	\$396.00
WELLS HIGH SPEED SPINDLES FOR BEGO &	NIRANIU	JM HIGH SPEED ALLOY GRINDERS	
	S036	For Bego High Speed Alloy Grinder • S120 Pulley for flat belt • S141 Pulley Shield • Four 6mm threaded mounting holes	\$396.00
	S042	For Niranium High Speed Alloy Grinder • S136 Pulley for double vee belt • S141 Pulley shield • S230 Smooth Mounting Tubes	\$396.00

Toll Free VOICE 1-800-233-0521 within U.S.


www.wellsdental.com

VOICE 707-937-0521 outside U.S.


Toll Free FAX 1-800-809-7553 within U.S.


Business Hours: Monday - Friday, 8am - 4:30pm

HIGH SPEED SPINDLES FOR WELLS GOLD CHROME FINISHING MACHINES

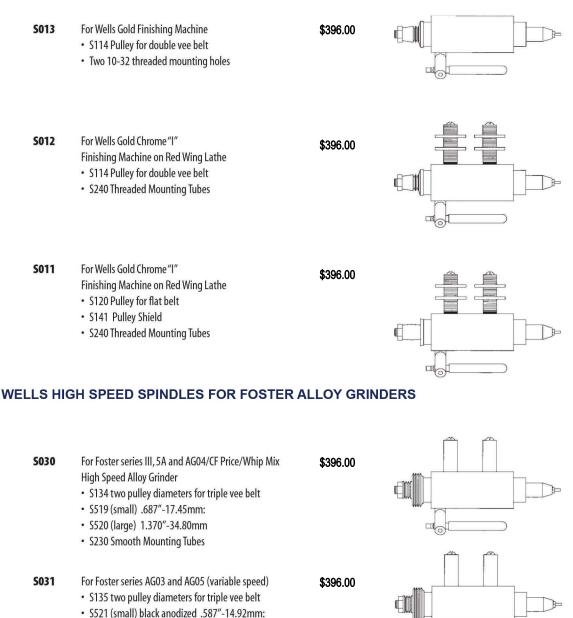

- S522 (large) black anodized 1.268"-32.20mm
- S230 Smooth Mounting Tubes
- **S028** For Foster Flat Belt High Speed Alloy Grinder
 - S130 and S132 two pulley diameters for flat belt
 S141 Pulley Shield
 - S230 Smooth Mounting Tubes
- S029 For Foster series PE3 and PEA High Speed Alloy Grinder • S130 and S128 two pulley diameters for a flat belt
 - S141 Pulley Shield
 - Housing for clamp type mounting

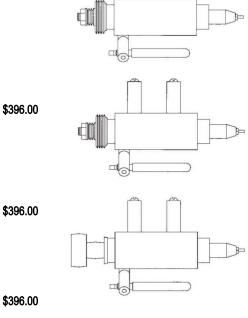

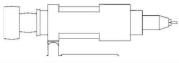

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.


Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

WELLS HIGH SPEED SPINDLE REPLACEMENT PARTS



Wells High Speed Spindles can be ordered to fit most alloy grinders. Please consult pages 10 & 11 for ordering information or call us for help. All spindles are shipped with S250 Accessory Kit and Users Manual.

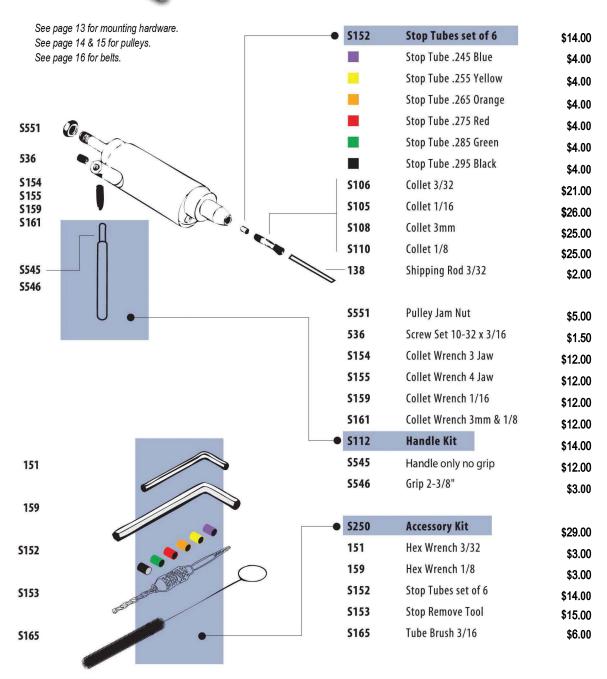

Toll Free VOICE 1-800-233-0521 within U.S.


www.wellsdental.com

VOICE 707-937-0521 outside U.S.

\$15.00

SPINDLE MOUNTING HARDWARE FOR WELLS HIGH SPEED SPINDLES

S220 Spindle Mounting Kit • Fastens S013 Wells High Speed Spindle to Wells Gold Chrome Finishing Machines: U011, U015, U020, U711 & U712

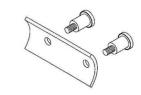

 S230 Mounting Tube Smooth - Pkg of 2
 Fastens S018, S028, S030, S031 & S042 Wells High Speed Spindles to Demco B1, Foster and Niranium Grinders



S231 Smooth Tube Rebuild Kit \$4.00

 S240 Mounting Tube Threaded - Pkg of 2
 Fastens S011 or S012 Wells high speen spindles to older Wells Gold Chrome "I" finishing machines






O O O aman

S241 Threaded Tube Rebuild Kit

\$8.00



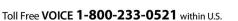

www.wellsdental.com


VOICE 707-937-0521 outside U.S.

9

Business Hours: Monday - Friday, 8am - 4:30pm

FINISHING MACHINES & COMPONENTS

See pages 10 and 11 for pulley configurations.

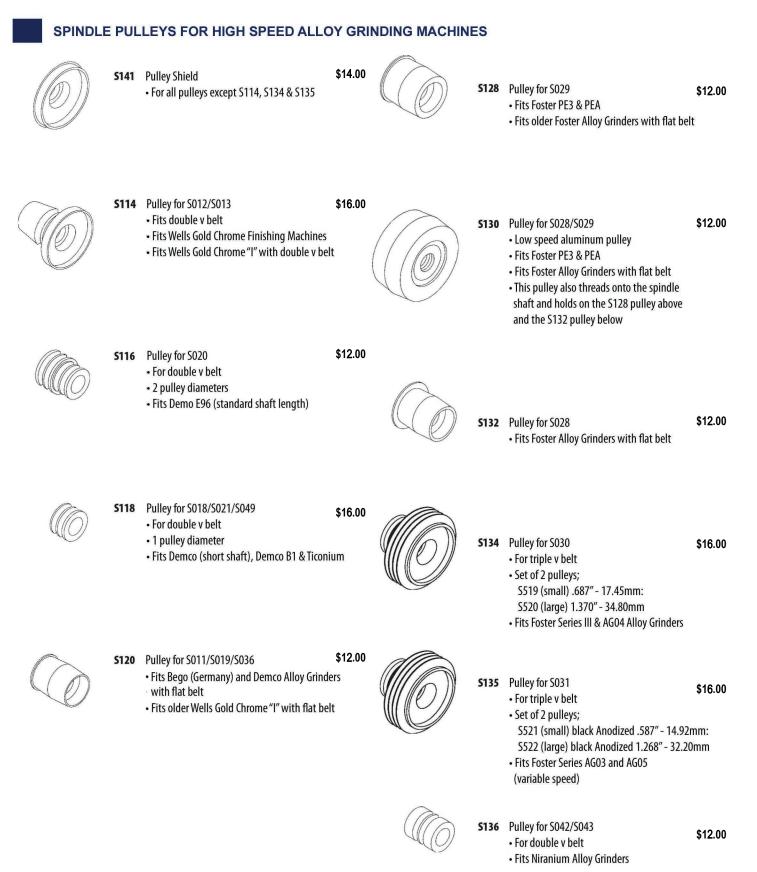

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

FAX 707-937-2809 outside U.S.

10

MOTOR PULLEYS FOR HIGH SPEED ALLOY GRINDNG MACHINES



S480 Motor Pulley 2V-V/Wells • 4-3/8" dia, flat edged pulley for double v belt

• For 1/2-20 threaded motor shaft

• Fits Wells R055, R056, R057 or R058 lathe with

Wells Gold Chrome Finishing Machine U712

\$46.00

- \$63.00
- 2 steps 3" and 3-1/2" dia, double v pulley
 For 1/2" dia motor shaft

S492 Motor Pulley 2V-2SP-2V/D/T

• Fits Demco, Ticonium Alloy Grinders



 S482
 Motor Pulley 2V-V/B/RW
 \$46.00
 S494
 Motor Pulley Flat 2SP RS CT/F
 \$63.00

 • 4-3/8" dia, flat edged pulley for double v belt
 • 2 steps 3-7/8" dia, crowned pulley

 • For 1/2" dia motor shaft
 for flat belt

 • Fits Red Wing Lathe with Wells Gold Chrome
 • For 1/2" dia motor shaft

 Finishing Machine
 • Fits Foster Alloy Grinders with flat belt

S484	Motor Pulley Flat Crown/RW	\$63.00	S495	Motor Pulley Flat 3-1/2" Dia	\$63.00
	• 3-7/8" dia, crowned pulley for flat belt			• 3-1/2" dia, crowned pulley for flat belt	
	For 1/2" dia motor shaft			For 1/2" dia motor shaft	
	• Fits Red Wing Lathe with older Wells Gold Chrome			Fits Demco Alloy Grinders with flat belt	
	"I" with flat belt				

S486	Motor Pulley Flat Crown/W/B	\$63.00	S496	Motor Pulley Flat 3-3/4" Dia	\$63.00
	 3-7/8" dia, crowned pulley for flat belt For 1/2" dia motor shaft 			 3-3/4" dia, crowned pulley for flat belt For 1/2" dia motor shaft 	
	Fits Baldor lathe with older Wells Gold Chrome			• Fits Demco Alloy Grinders with flat belt	
	"I" with flat belt				

Motor Pulley 2V-2V/D/TIC/NIR • 3-1/2" dia, double v pulley • For 1/2" dia motor shaft • Fits Demco, Ticonium and Niranium Grinders	\$63.00	S498	Motor Pulley 1V2SP/Demco • 2 steps 3-1/8" and 3-3/8" dia, single v pulley • For 1/2" dia motor shaft • Fits Demco Alloy Grinders	\$63.00
Fits Demco, liconium and Niranium Grinders			Fits Demco Alloy Grinders	


Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

FINISHING MACHINES & COMPONENTS

V BELTS FOR HIGH SPEED ALLOY GRINDING MACHINES

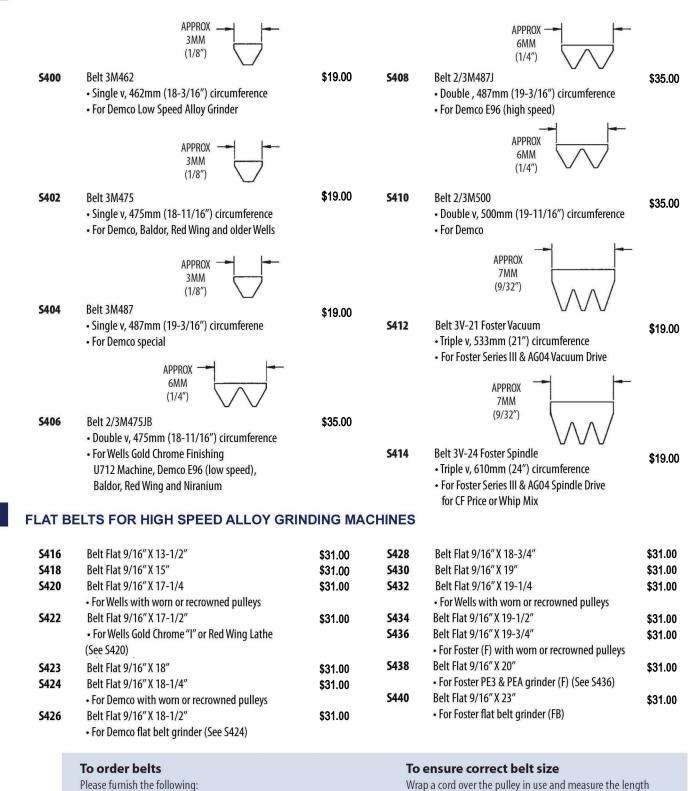

1. Model and manufacturer of the alloy grinder

- 2. Serial number of the spindle
- 3. The type of belt (flat belt, 1, 2 or 3 v belt)
- 4. The circumference of the belt (total length)

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

either in inches or millimeters.

FAX 707-937-2809 outside U.S.

12

SUPER QUICK CHUCKS MOUNTED ON LATHES, DIRECT DRIVEN



U510 Quick Chuck on 1/3Hp lathe 115V 60Hz \$1479.00 1725/3450 RPM For "Heavy Duty" grinding & polishing



U523 Quick Chuck on 1/3Hp lathe with Light Shield \$1629.00 1725/3450 RPM For "Heavy Duty" grinding & polishing

R057 Wells 1/3Hp Lathe 115V 60Hz \$1114.00 1725/3450 RPM Ready to accept Quick Chuck Q015 Clutch is Installed For "Heavy Duty" grinding & polishing





U610 Quick Chuck on right & left side on \$1949.00 1/3 hp lathe, 115V 60 Hz 1725/3450 RPM For "Heavy Duty" grinding & polishing

SUPER QUICK CHUCK SPECIFICATIONS

Collet Sizes:	integrate	d 1/4" and 3/32"
Operating speeds:	1725 and	1 3450 RPM
Unit dimensions:	U510 U523	approx. 8" X 9" X 18″ approx. 8" X 11" X 24"
Shipping weight::	U510 U523	50 lbs 52 lbs
Shaft height::	5 1/4"	



U629 Semi-Perm on right & left side on 1/3 hp lathe, 115V 60 Hz 1725/3450 RPM For "Heavy Duty" polishing

\$1662.00

Other Lathes & Quick Chuck combinations are available. Please call for more information

Toll Free VOICE 1-800-233-0521 within U.S.


www.wellsdental.com

VOICE 707-937-0521 outside U.S.

Toll Free FAX 1-800-809-7553 within U.S.

FAX 707-937-2809 outside U.S.

13

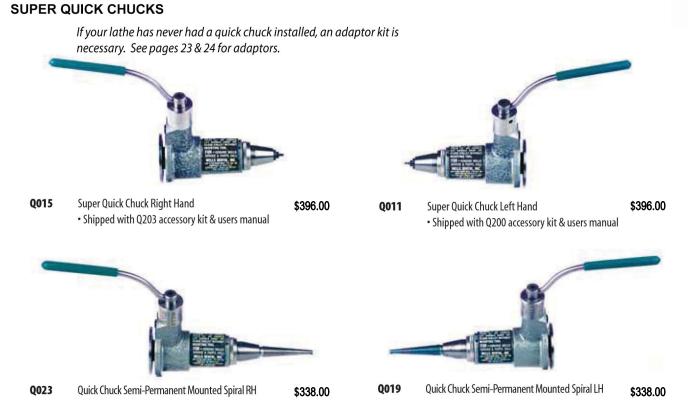

Toll Free VOICE 1-800-233-0521 within U.S.


www.wellsdental.com

VOICE 707-937-0521 outside U.S.

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

WELLS SUPER QUICK CHUCK REPLACEMENT PARTS & TOOLS

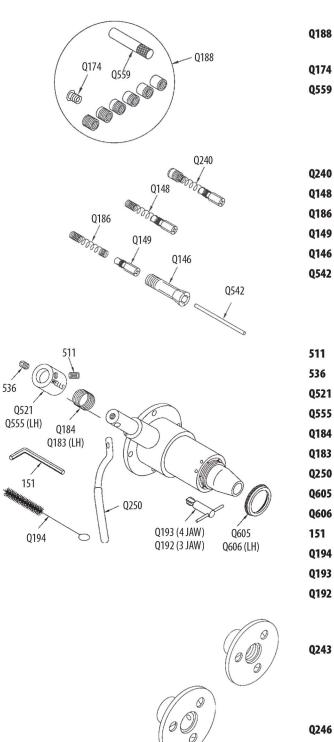

8	Stops – set of 6 • With stop lock screw & test rod	\$18.00
4	Stop Lock Screws – Pkg. of 2	\$4.00
9	Test Rod 1/4"	\$4.00
	 For fitting a stop to adjust the collet diameter 	

10	Special Collet Complete	\$28.00
18	Special Collet With Spring	\$26.00
36	Pop-Out Spring – Pkg. of 2	\$5.00
19	Special Collet	\$23.00
16	Master Collet	\$26.00
12	Shipping Rod 3/32" X 2-1/8"	\$2.00

I	Screw set 10-32" x 1/4"	\$1.50
6	Screw Set 10-32" X 3/16"	\$1.50
21	Number Cap RH	\$24.00
55	Number Cap LH	\$24.00
34	Clutch Spring RH	\$13.00
33	Clutch Spring LH (painted red)	\$13.00
50	Handle with Set Screw	\$15.00
)5	Dust Cover RH	\$16.00
)6	Dust Cover LH	\$18.00
I	Hex Wrench 3/32"	\$3.00
94	Tube Brush 3/8"	\$6.00
93	Collet Wrench 4 Jaw	\$13.00
92	Collet Wrench 3 Jaw	\$13.00

}	Clutch Wells Threaded Complete	\$41.00
	 For Quick Chuck on Wells R057 Lathe 	
	Clutch threads onto shaft of lathe	

5 Clutch Baldor/Red Wing Complete • For Quick Chuck on Baldor or Red Wing • Clutch held on with set screw

Clutch Lining with Screws

For all Wells Super Quick Chucks

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

Q150

Ø

Ø

0

VOICE 707-937-0521 outside U.S.


\$22.00


Toll Free FAX 1-800-809-7553 within U.S.




Q203	Accessory Kit RH • For Q015 Quick Chuck Right Hand	\$84.00	See the following illustrations
	Q134 Spiral Chuck RH (1 included in kit)		Q134 Q126
	Q126 Arbor Band Chuck 3/4 RH (1)		\sim
	• Q154 Mandrel with 1-64 screw RH (2)		
	Q142 Stone Chuck Tooth & Knife RH (2)		
	Q188 Stops - set of 6		
	(includes Stop Lock Screw and Test Rod)		Q188
	• Q194 Tube Brush 3/8		
	• Q193 Collet Wrench 4 jaw		
	• 151 Hex Wrench 3/32		
	• Q169 Installation Screws — Pkg. of 4		
Q200	Accessory Kit LH • For Q011 Quick chuck Left Hand	\$94.00	
	• Q132 Spiral Chuck LH (1 included in kit)		Q140
	• Q124 Arbor Band Chuck 3/4 LH (1)		
	• Q152 Mandrel with 1-64 screw LH (2)		
	• Q140 Stone Chuck Tooth & Knife LH (2)		151
	Q188 Stops - set of 6		
	(includes Stop Lock Screw and Test Rod)		
	Q194 Tube Brush 3/8		Q169 (Manual C
	Q193 Collet Wrench 4 jaw		
	• 151 Hex Wrench 3/32		
	Q169 Installation Screws — Pkg. of 4		
L013	Light Shield - Quick Chuck RH • Light fixture and tempered glass shield • Installs with right hand Quick Chuck • Light bulbs are not included	\$165.00	
L014	Light Shield - Quick Chuck LH Light Fixture and tempered glass shield Installs with left hand Quick Chuck Light bulbs are not included 	\$180.00	
L110	Glass Shield for Wells • Tempered glass for L011, L013 & L014 • 4″ X 6″	\$23.00	
Q104	Collet Shield RH • Fits right hand Quick Chuck with knurled stainless steel dust cover (ring) only.	\$19.00	
Q103	 Collet Shield LH Fits left hand Quick Chuck with knurled stainless steel dust cover (ring) only. 	\$28.00	

See the following illustrations for details of individual accessories.

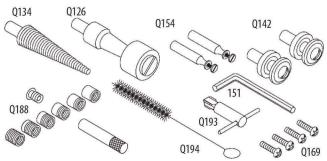

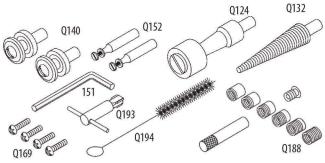

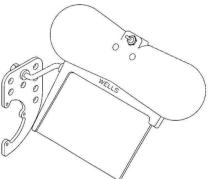

New

This non-rotating collet shield covers the collet body to provide a steady rest for more comfortable operation of your Wells Super Quick Chuck. It also helps protect rubber gloves.


Toll Free VOICE 1-800-233-0521 within U.S.


www.wellsdental.com


VOICE 707-937-0521 outside U.S.


Business Hours: Monday - Friday, 8am - 4:30pm

SUPER QUICK CHUCK ACCESSORIES

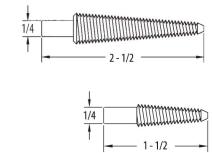


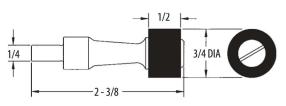


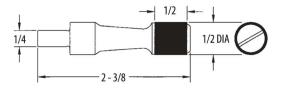


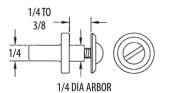

See pages 27- 40 for consumables

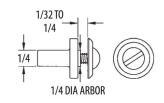

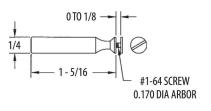

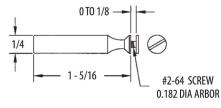

Q134	Spiral Chuck RH • For brushes and buffs	\$15.00
Q132	Spiral Chuck LH • For use on the left side of the lathe	\$15.00
Q131	Spiral Chuck Short RH • For brushes and buffs	\$15.00
Q133	Spiral Chuck Short LH For use on the left side of the lathe 	\$15.00
Q126	Arbor Band Chuck 3/4 RH • Expanding drum for 3/4" dia. abrasive bands	\$15.00
Q124	Arbor Band Chuck 3/4 LH • For use on the left side of the lathe	\$15.00
Q122	Arbor Band Chuck 1/2 RH • Expanding drum for 1/2" dia. abrasive bands	\$15.00
Q120	Arbor Band Chuck 1/2 LH • For use on the left side of the lathe	\$15.00
Q142	Stone Chuck Tooth & Knife RH • For wheels and stones 1/4" to 3/8" thick • 1/4" dia. arbor	\$15.00
Q140	Stone Chuck Tooth & Knife LH For use on the left side of the lathe 	\$15.00
Q138	 Stone Chuck Thin Econo RH For Econo Cutters, wheels and stones from 1/32" to 1/4" thick 1/4" dia. arbor 	\$15.00
Q136	Stone Chuck Thin Econo LH For use on the left side of the lathe 	\$15.00
Q154	Mandrel with 1-64 screw RH • For separating & finishing discs • For diamond discs & heatless wheels • 0.170" dia. arbor	\$15.00
Q152	Mandrel with 1-64 screw LH For use on the left side of the lathe 	\$15.00
Q158	Mandrel with 2-64 screw RH • For separating & finishing discs • For diamond discs & heatless wheels • 0.182" dia. arbor	\$15.00
Q156	Mandrel with 2-64 screw LH • For use on the left side of the lathe.	\$15.00

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

17

SUPER QUICK CHUCK ACCESSORIES

SUPER Q	UICK CHUCK ACCESSORIES		1/16 DIA —
Q106	Burr Adapter 1/16 • For holding 1/16" dia.shank in 1/4" chuck	\$16.00	
Q108	Burr Adapter 3/32 • For holding 3/32" dia. shank in 1/4" chuck	\$16.00	3/32 DIA
Q110	Burr Adapter 1/8 • For holding 1/8" dia.shank in 1/4" chuck	\$16.00	1/8 DIA 1/4
Q112	Burr Adapter 3MM • For holding 3mm dia.shank in 1/4" chuck	\$16.00	3MM DIA 1/4 3/4
Q130	Sandpaper Chuck Slotted mandrel for holding sandpaper roll • Roll is held in place with waxed string	\$15.00	1/4 DIA
Q128	Field Sandpaper Chuck RH • For "Field" abrasive spiral rolls	\$15.00	0.435 DIA MANDREL 1/4 1-3/8 0.435 DIA MANDREL
Q127	Field Sandpaper Chuck LH • For use on the left side of the lathe	\$15.00	
Q198	Field Sandpaper Chuck Long RH • For "Field" abrasive spiral rolls	\$15.00	0.435 DIA MANDREL
Q197	Field Sandpaper Chuck Long LH • For use on the left side of the lathe	\$15.00	
Q118 Renlacen	Arbor - Yates Burr • For "Yates" high speed steel burrs. nent parts for these accessories are available. See the	\$15.00	1/4 - 28 THREADS 1/4
nepracen	in parts for these accessories are available. See the	contraction church	

Replacement parts for these accessories are available. See the Quick Chuck User's Manual for part numbers or call us for assistance.


Toll Free VOICE 1-800-233-0521 within U.S.	www.wellsdental.com	VOICE 707-937-0521 outside U.S.
Toll Free FAX 1-800-809-7553 within U.S.	Business Hours: Monday - Friday, 8am - 4:30pm	FAX 707-937-2809 outside U.S.

POLISHING LATHES

R051



LATHE ACCESSORIES

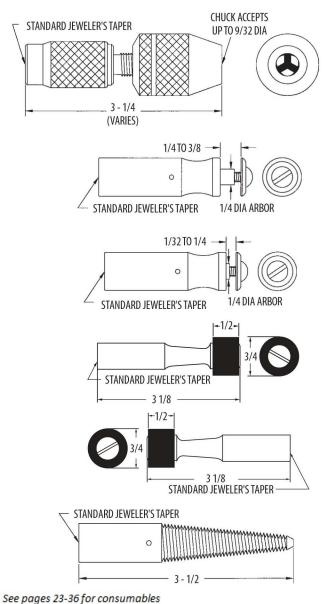

R051 (Shown)

Baldor 380T Lathe 1/3 Hp

• "Heavy Duty" laboratory polishing lathe

- Two Speeds, 1725 and 3450 rpm
- 1/3 Hp, 115 Volt, 60 Hz, 1 phase motor
- Standard Jeweler's taper (1/4" per ft.) with chuck ejector on both ends The 1/3 hp lathe is also available in 100 or 230 volts Shipped from manufacturer NON-RETURNABLE

R054 (Not Shown)

\$829.00

- Baldor 340 Lathe 1/4 Hp

 "Light Duty" laboratory polishing lathe
 - Two Speeds, 1800 and 3600 rpm
- 1/4 Hp, 115 Volt, 60 Hz, 1 phase motor
- Standard Jeweler's taper (1/4" per ft.) with chuck ejector on both ends Shipped from manufacturer NON-RETURNABLE

R131	3 Jaw Chuck RHFor tools with up to 9/32" (7mm) shankFor use on the right side of the lathe	\$69.00
R130	3 Jaw Chuck LHFor tools with up to 9/32" (7mm) shankFor use on the left side of the lathe	\$69.00
R133	Stone Chuck Taper Shaft RH • For wheels and stones 1/4" to 3/8" thick • 1/4" dia. arbor	\$15.00
R132	Stone Chuck Taper Shaft LH • For use on the left side of the lathe	\$15.00
R135	 Thin Econo Taper Shaft RH For Econo Cutters, wheels and stones from 1/32" to 1/4" thick 1/4" dia. arbor 	\$15.00
R134	Thin Econo Taper Shaft LH • For us on the left side of the lathe	\$15.00
R148	Arbor Band Taper Shaft 3/4 RH • Expanding drum for 3/4 dia. abrasive bands	\$15.00
R147	Arbor Band Taper Shaft 3/4 LH • For use on the left side of the lathe	\$15.00
R137	Spiral Taper Shaft RH Stainless Steel • For brushes and buffs	\$15.00
R136	Spiral Taper Shaft LH Stainless Steel • For use on the left side of the lathe.	\$15.00
R139	Spiral Brass Taper Shaft RH • Solid brass for brushes and buffs	\$18.00
R138	Spiral Brass Taper Shaft LH • For use on the left side of the lathe	\$18.00

, , ,

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com


VOICE 707-937-0521 outside U.S.


19

\$959.00

Business Hours: Monday - Friday, 8am - 4:30pm

SUPER QUICK CHUCK ADAPTORS

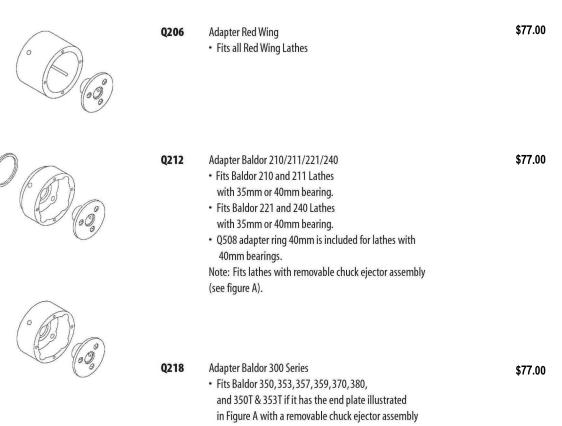

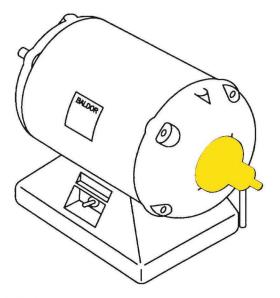

Lathe Condition:

The lathe should be in good running condition. If it has noisy bearings, a worn switch, starting problems or excessive end play (greater than 1/32"), the condition should be corrected before proceeding with the installation. The condition of the tapered shaft is not important because it is not used.

FIGURE A


Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

SUPER QUICK CHUCK ADAPTORS

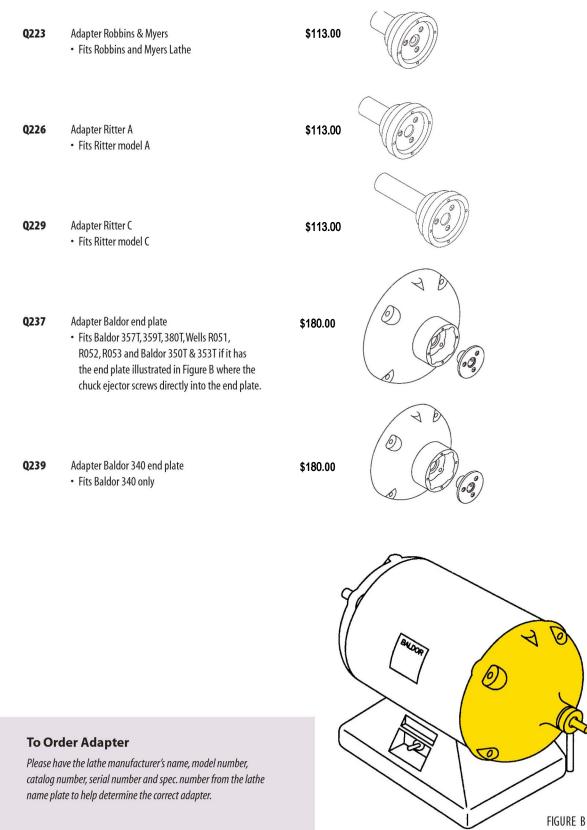

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

LATHE ACCESSORIES

P011	Precision Chuck 3/8" taper • Quick change burr chuck • For tools with 3/32" dia shank • Tools may be changed while lathe is running • Fits standard jeweler's taper shaft	\$105.00	ACCEPTS 3/32 DIA SHANK
P012	 Precision Chuck 1/4 Shaft Quick change burr chuck For tools with 3/32" dia shank Tools may be changed while lathe is running Fits any chuck that accepts a 1/4" dia.shaft 	\$105.00	ACCEPTS 3/32 DIA SHANK
R142	 Shaft Guard RH /Wells/B Fits Wells and Baldor polishing lathes Covers the shaft on the right side of the lathe Identified by one grove on its exterior 	\$28.00	
R141	 Shaft Guard LH/Wells/B Fits Wells and Baldor polishing lathes Covers the shaft on the left side of the lathe Identified by three groves on its exterior 	\$28.00	R142
R144	Shaft Guard RH/RW#2 • Fits Red Wing lathe with one piece chuck remover • Covers the shaft on the right side of the lathe • Shaft guard is male threaded • Identified by two groves on its exterior	\$28.00	
R143	 Shaft Guard LH/RW#2 Fits Red Wing lathe with one piece chuck remover Covers the shaft on the left side of the lathe Shaft guard is male threaded Identified by five groves on its exterior 	\$28.00	R144 Male threads shown
R146	 Shaft Guard RH/RW#1 Fits Red Wing lathe with two piece chuck remover Covers the shaft on the right side of the lathe Shaft guard is female threaded Identified by two groves on its exterior 	\$28.00	R146 Female threads shown
R145	 Shaft Guard LH/RW#1 Fits Red Wing lathe with two piece chuck remover Covers the shaft on the left side of the lathe. Shaft guard is female threaded Identified by three groves on its exterior The Shaft Guard satisfies OSHA regulations for covering	\$28.00 g rotating shafts.	

22

Toll Free VOICE 1-800-233-0521 within U.S. Toll Free FAX 1-800-809-7553 within U.S. www.wellsdental.com Business Hours: Monday - Friday, 8am - 4:30pm VOICE 707-937-0521 outside U.S.

BURRS

Our principle is to provide the optimum solution in terms of quality and cost. One of our solutions is the TN-Titanium Nitride coated carbide burr. These TN coated carbide burrs last at least three times long as uncoated burrs before resharpening. All our carbide burrs are available with TN Coating.







		UNCOATED	COATED
UC-SA-61-A	Burr (diamond cut)	\$19.00	
TN-SA-61-A	Burr (diamond cut)		\$22.00
UC-SF-61	Burr (diamond cut)	\$19.00	
TN-SF-61	Burr (diamond cut)		\$22.00
UC-SA-61	Burr (diamond cut)	\$19.00	
TN-SA-61	Burr (diamond cut)		\$22.00
UC-SG-61	Burr (diamond cut)	\$19.00	
TN-SG-61	Burr (diamond cut)		\$22.00
UC-SC-61	Burr (diamond cut)	\$19.00	
TN-SC-61	Burr (diamond cut)		\$22.00
UC-SM-61	Burr (diamond cut)	\$19.00	
TN-SM-61	Burr (diamond cut)		\$22.00
UC-SN-61	Burr (diamond cut)	\$19.00	
TN-SN-61	Burr (diamond cut)		\$22.00
UC-SD-61	Burr (diamond cut)	\$19.00	
TN-SD-61	Burr (diamond cut)		\$22.00
UC-SE-61	Burr (diamond cut)	\$19.00	
TN-SE-61	Burr (diamond cut)		\$22.00
UC-53D	Burr (regular cut 3/16" Ball)	NA	
TN-53D	Burr (regular cut 3/16" Ball)		\$28.00
UC-51A	Burr (regular cut)	\$23.00	
TN-51A	Burr (regular cut)		\$28.00
UC-61-81A	Burr (regular cut)	\$23.00	
TN-61-81A	Burr (regular cut)		\$28.00
UC-73C	Burr (regular cut)	\$23.00	
TN-73C	Burr (regular cut)		\$28.00
UC-88A	Burr (regular cut)	\$23.00	
TN-88A	Burr (regular cut)		\$28.00
UC-83E	Burr (regular cut)	\$23.00	
TN-83E	Burr (regular cut)		\$28.00


Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

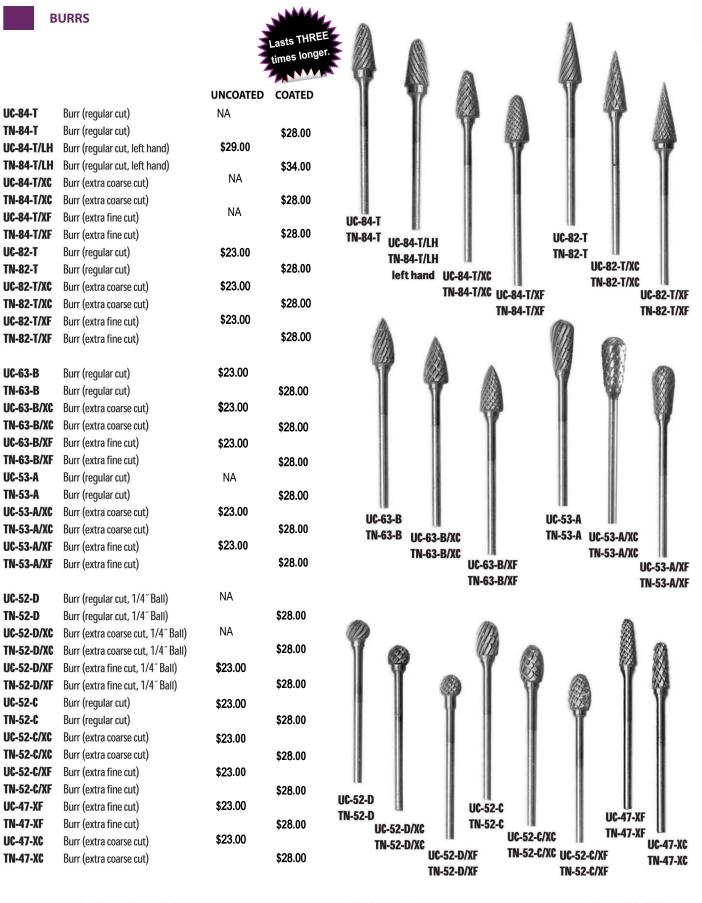

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm





UC-41P	Burr (diamond cut)	UNCOATED \$19.00	COATED
TN-41P	Burr (diamond cut)	\$19.00	\$22.00
UC-44C	Burr (diamond cut)	\$19.00	722.00
TN-44C	Burr (diamond cut)	¥10.00	\$22.00
UC-31P	Burr (diamond cut)	NA	ψΖΖ.00
TN-31P	Burr (diamond cut)		\$22.00
UC-32P	Burr (diamond cut)	\$19.00	Ψ ΞΙΟΟ
TN-32P	Burr (diamond cut)		\$22.00
UC-33C	Burr (diamond cut)	\$19.00	,
TN-33C	Burr (diamond cut)		\$22.00
UC-71G	Burr (diamond cut)	\$19.00	
TN-71G	Burr (diamond cut)		\$22.00
UC-88-A-4	Burr (regular cut)	\$22.00	
TN-88-A-4	Burr (regular cut)		\$29.00
UC-88-B-4	Burr (regular cut, end cut)	\$22.00	
TN-88-B-4	Burr (regular cut, end cut)		\$29.00
UC-88-C-4	Burr (regular cut)	\$22.00	
TN-88-C-4	Burr (regular cut)		\$29.00
UC-88-T-4	Burr (regular cut)	NA	
TN-88-T-4	Burr (regular cut)		\$29.00
UC-A3/8	Burr (regular cut)	NA	
TN-A3/8	Burr (regular cut)		\$55.00
UC-A3/8 LH	Burr (regular cut, left hand)	NA	
TN-A3/8 LH	Burr (regular cut, left hand)		\$64.00
UC-B3/8	Burr (regular cut)	\$46.00	
TN-B3/8	Burr (regular cut)		\$55.00
UC-B3/8 DIA	Burr (diamond cut)	\$46.00	
TN-B3/8 DIA	Burr (diamond cut)	¢40.00	\$55.00
UC-C3/8 TN-C3/8	Burr (regular cut)	\$46.00	
UC-D3/8	Burr (regular cut)	\$46.00	\$55.00
UC-D3/8 TN-D3/8	Burr (regular cut)	\$40.00	AFE 00
UC-D3/8 LH	Burr (regular cut) Burr (regular cut, left hand)	\$55.00	\$55.00
TN-D3/8 LH	Burr (regular cut, left hand)	400.00	¢04.00
UC-E3/8	Burr (regular cut)	\$46.00	\$64.00
TN-E3/8	Burr (regular cut)	ψτ0.00	\$55.00
UC-F3/8	Burr (regular cut)	\$46.00	400.00
TN-F3/8	Burr (regular cut)	¥ 10100	\$55.00
11110/0	ban nogalal bay		φυυ.00



Business Hours: Monday - Friday, 8am - 4:30pm

VOICE 707-937-0521 outside U.S.

asts imes longe

BURRS				asts THF mes long
For gross remova	al of plaster	UNCOATED	COATED	
UC-84-T/SP	Burr (alum cut)	\$31.00		
TN-84-T/SP	Burr (alum cut)		\$36.00	
UC-88-A/SP	Burr (alum cut)	\$31.00		ß
TN-88-A/SP	Burr (alum cut)		\$36.00	
UC-A3/8 SP	Burr (alum cut)	\$54.00		
TN-A3/8 SP	Burr (alum cut)		\$64.00	7
UC-A1/2 SP	Burr (alum cut)	\$69.00		
TN-A1/2 SP	Burr (alum cut)		\$82.00	
UC-A1/2	Burr (regular cut)	\$58.00		
TN-A1/2	Burr (regular cut)		\$70.00	
UC-A1/2 LH	Burr (regular cut, left hand)	NA		
TN-A1/2 LH	Burr (regular cut, left hand)		\$82.00	
UC-B1/2	Burr (regular cut)	\$58.00		
TN-B1/2	Burr (regular cut)		\$70.00	
UC-B1/2DIA	Burr (diamond cut)	\$60.00		
TN-B1/2DIA	Burr (diamond cut)		\$73.00	
UC-C1/2	Burr (regular cut)	\$58.00		
TN-C1/2	Burr (regular cut)		\$70.00	
UC-D1/2	Burr (regular cut)	NA		UC-
TN-D1/2	Burr (regular cut)		\$70.00	TN-
UC-D1/2 LH	Burr (regular cut, left hand)	\$70.00		
TN-D1/2 LH	Burr (regular cut, left hand)	NA	\$82.00	
UC-E1/2	Burr (regular cut)	INA		
TN-E1/2	Burr (regular cut)		\$70.00	
UC-F1/2	Burr (regular cut)	\$58.00		
TN-F1/2	Burr (regular cut)		\$70.00	





TN-A1/2 SP

TN-A3/8 SP

UC-A1/2 LH UC-D1/2 LH UC-B1/2 DIA UC-D1/2 UC-A1/2 TN-A1/2 LH UC-B1/2 UC-C1/2 TN-D1/2 LH UC-E1/2 UC-F1/2 TN-A1/2 TN-B1/2 TN-B1/2 DIA TN-C1/2 TN-D1/2 **TN-E1/2** TN-F1/2 left hand left hand **Diamond Cut**

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

26

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

MODEL TRIMMER WHEELS


Model Trimmer Wheels for Ray Foster model trimmers as well as other brands.

- 10" Diameter
- 1″ Hole
- 5/8" Drive Hole

C16TB	Model Trimmer Wheel (Coarse)	\$29.00 each
C20TB	Model Trimmer Wheel (Medium Coarse)	\$29.00 each
C30TB	Model Trimmer Wheel (Fine)	\$29.00 each
C60TB	Model Trimmer Wheel (Extra Fine)	\$29.00 each



ECONO CUTTERS

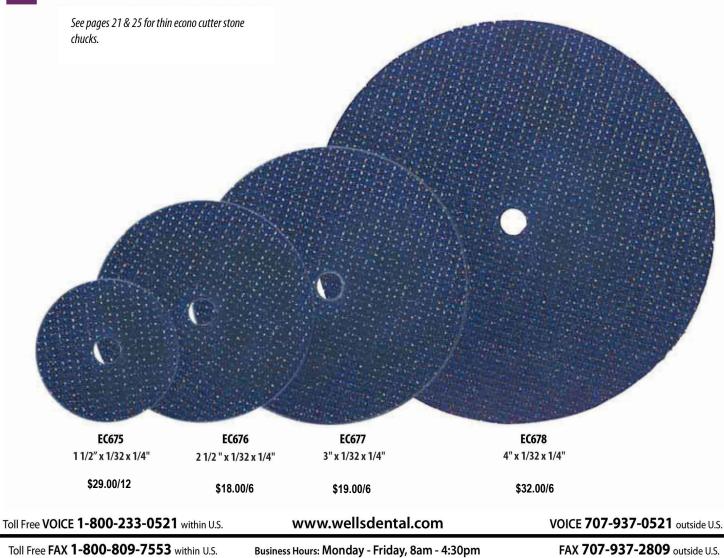

ALUMINUM OXIDE ROUGHING DISCS

Pure aluminum oxide disc, non-contaminating, long-lasting, very fast cutting, and odorless.



RD620 35mm x 1.7mm \$36.50/100

RD621 32mm x 2.2mm \$36.50/100

CW310

1 1/2" x .040"

38.8 x 1.1mm

\$25.00/100



RD622 22mm x 3.5mm 36.50/100

KEYSTONE CUT-OFF WHEELS

These Cut-Off Wheels are strong and durable. Made with pure aluminum oxide provide non-contaminting fast free cutting of all metals. The wheels come in a reddish-brown color and have a max of 25,000 rpm.



CW320 1 1/4" x .062" 31.8 x 1.6mm

\$25.00/100

SEPARATING DISCS



\$46.25/100



\$46.25/100

Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm

FAX 707-937-2809 outside U.S.

38.8 x .63mm

\$25.00/100

28

REPAIRS

FACTORY REPAIR

Our service department assures prompt repairs of all Wells products. We have a one to two day turn around time which will get you back into production quickly. There is no return authorization necessary.

When we receive your equipment for repair it is thoroughly checked for worn or damaged parts. Wells equipment can always be rebuilt to factory specifications with a fully reinstated prorated warranty. Your equipment will come back to you just like new at a significant savings over buying new equipment.

BEFORE YOU SEND YOUR EQUIPMENT FOR REPAIR:

1. Consult the technical bulletin or user's manual that came with your equipment. Problems are often solved by simple maintenance procedures. All technical bulletins and user's manuals can be found on our web site www.wellsdental.com. Simply click on "documents" and click on the appropriate manual.

2. Call our toll-free number 1-800-233-0521 and ask for technical support. Our experience may save you extra work and delay.

3. If the equipment needs service beyond your means, ship it directly to us and we will be happy to repair and return it. No return authorization is necessary. We encourage you to send Wells equipment directly to us.

4. UPS is the only delivery company that comes to our location daily. USPS is another alternative. Other major carriers do not deliver to our rural location. To avoid delays please ship UPS to our street address. If you prefer to use the postal service please mail to our post office box.

5. Ship only Wells products to us with your name, your company name, UPS deliverable shipping address (no P.O. boxes) and phone number along with a brief description of the problem and any special instructions. In order to keep our turn around repair service quick we will not call you with an estimate unless you specifically ask in writing. Maximum repair prices are well under the cost of new equipment. Estimates will take an additional one to two days. If you do not have an account with us, your items will be shipped back to you UPS ground COD. If you would like to avoid COD charges, we encourage you to include in your note your credit card number and expiration date.


EQUIPMENT EXCHANGE

If you cannot be without your equipment for the one or two days it takes us to repair your item our exchange service is available. We can send you a rebuilt item with the understanding that you will return your old item to us. Enclosed with the packing list on top of the box you will find a UPS prepaid RS (return service) label. Simply enclose your old equipment in the same box and affix the prepaid return label. You can give the package to any UPS driver and the package will be returned to us. The equipment exchanged will be invoiced at full retail price which serves as a deposit. Upon receipt of your old item, appropriate credit will be issued for the difference between the retail price and the exchange price. No credit will be issued for exchanges not returned within 30 days of invoice date.

TRADE-IN

We offer a trade-in service for equipment from other manufacturers. If you would like to update to a Wells High Speed Spindle, a Wells Super Quick Chuck or a Wells Handpiece and/or Arm we offer a discount on new equipment.



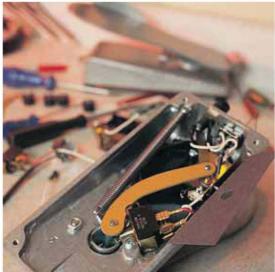



Toll Free VOICE 1-800-233-0521 within U.S.

www.wellsdental.com

VOICE 707-937-0521 outside U.S.

Toll Free FAX 1-800-809-7553 within U.S.

Business Hours: Monday - Friday, 8am - 4:30pm





MAILING ADDRESS:

WELLS DENTAL INC PO BOX 106 COMPTCHE, CA 95427-0106 USA

SHIPPING ADDRESS:

WELLS DENTAL INC 5860 FLYNN CREEK RD COMPTCHE, CA 95427-9500

